




Flexible. Customizable. Upgradable.

Beds for modern healing spaces.



ACURA

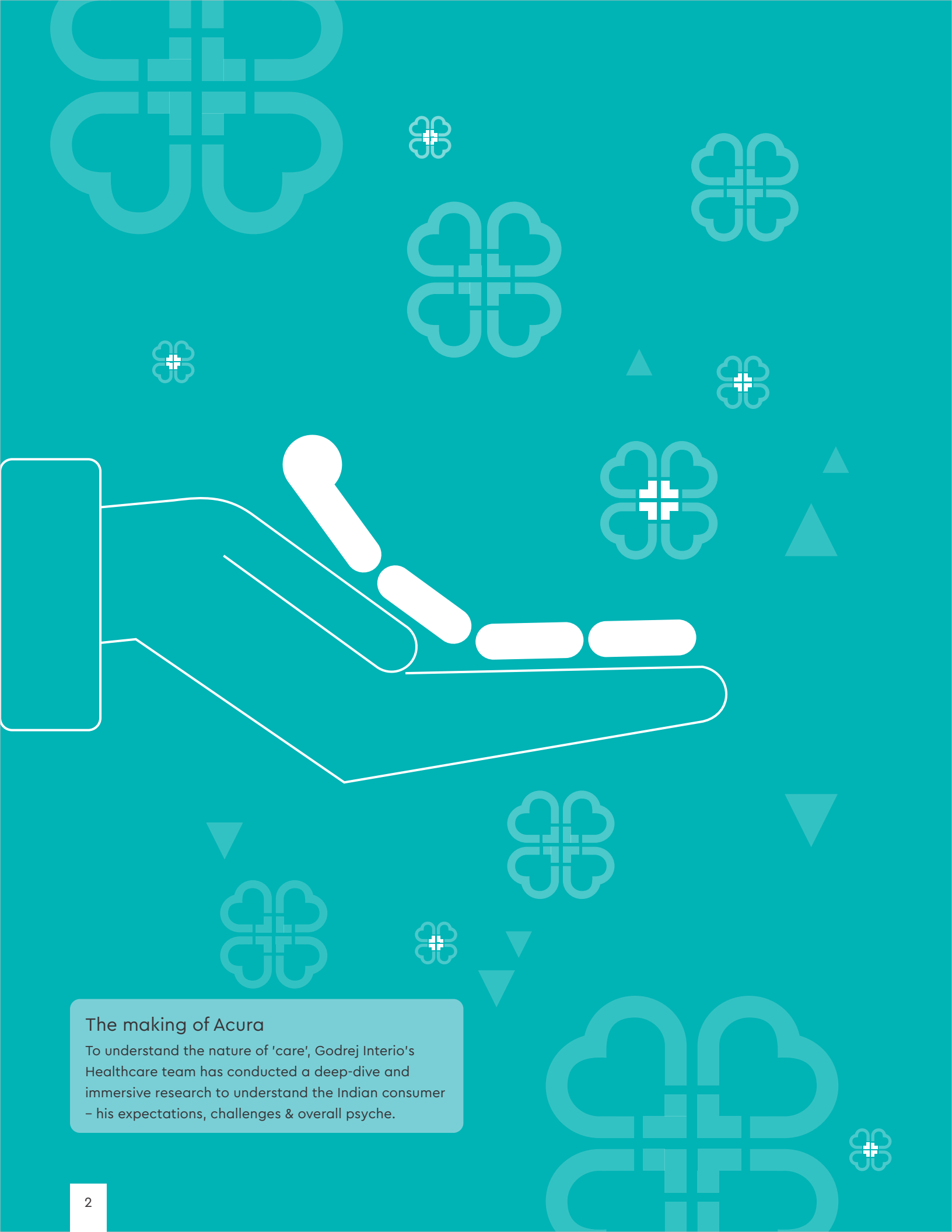

The making of Acura

To understand the nature of 'care', Godrej Interio's Healthcare team has conducted a deep-dive and immersive research to understand the Indian consumer – his expectations, challenges & overall psyche.

ACURA

Where design, innovation and healthcare converge

A platform-based range of hospital beds, Acura is designed to offer care, comfort and ease of operation at every stage. This range gives you 'Buy what you need' options for varying needs of different hospital environments.




Configured to your needs

Allows you to choose from a menu of options to configure bed according to your needs.




Bed Upgrades: Now Onsite & Easy

Acura beds come with the flexibility of converting manual functions to motorized ones with help of retrofit electronics kit on-site without making any structural changes in the bed. With this retrofit electronic kit, the manual functions of Back Rest, Leg Rest & Height Adjustment can be motorized at site in a few hours, thereby minimizing downtime, customer spend and associated environmental impact.

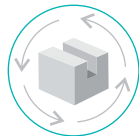


A more efficient product lifecycle

No reworks, no replacements. Acura offers the customer a financially more attractive solution to shift from manual to motorised. This solution will eliminate entire process of new product replacement; while saving on manufacturing, transportation and installment resources.

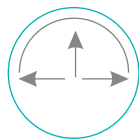

Range of finishes & functionalities

From the initial choice of finishes to a range of functionalities, Acura is designed to give you choice. The result is a bed that can adapt to the needs of an expanding healthcare facility across years.

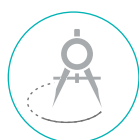

An example of Innovation in design engineering

Godrej Interio designers and engineers have created a new high performance adaptive bed platform, led by innovation in design engineering and implementation processes. The onsite upgradability feature in Acura has been patented.

Acura Beds


The smarter way to upgrade.




3 unique platforms
4 varieties of sideboard configurations
Onsite upgradability from manual to motorised

What makes the Acura 'Manual to Motorised' Range unique is the facility provided in the bed frame to shift from the manual crank adjustment to motorized adjustments, easily and quickly, onsite. This product range provides you a wide choice of hospital beds, that offer upgrade features making it the preferred choice for fast growing healthcare spaces.

Buy as you need, upgrade at your convenience

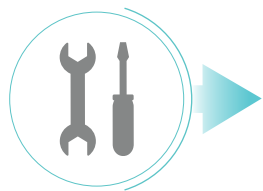

Start with bed having manual features

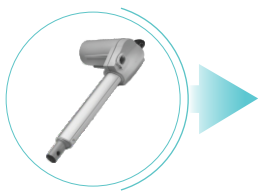

Purchase Acura Actuator System














Godrej Interio service technician installs the actuator on the bed, at your site









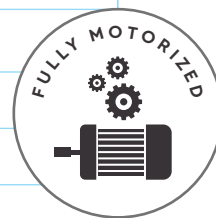



Bed upgraded from manual to motorised


Acura Prime

FUNCTION	BASIC BED	AT TIME OF PURCHASE
Back rest	Not applicable 	Motorized Functions 
Leg rest	Not applicable 	Motorized Functions 
Height Adjustment	Not applicable 	Motorized Functions 
Trendelberg	Not applicable 	Motorized Functions 
Reverse Trendelberg	Not applicable 	Motorized Functions 





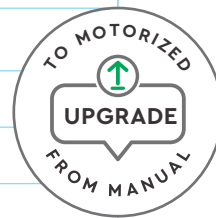


Acura Standard

FUNCTION	BASIC BED	POST UPGRADATION
Back rest	Manual Function 	Motorized Functions 
Leg rest	Manual Function 	Motorized Functions 
Height Adjustment	Manual Function 	Motorized Functions 
Trendelberg	Manual Function 	Manual Function 
Reverse Trendelberg	Manual Function 	Manual Function 



Acura Vital

FUNCTION	BASIC BED	POST UPGRADATION
Back rest	Manual Function 	Motorized Functions 
Leg rest	Manual Function 	Motorized Functions 
Height Adjustment	Not available	Not available
Trendelberg	Not available	Not available
Reverse Trendelberg	Not available	Not available

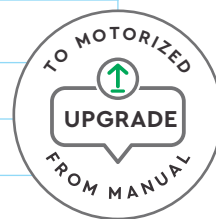

Perfect for responsive healthcare spaces



Simple, Safe
and Convenient



Bed function convertibility
from manual to motorized



Compliant to
International Standards



Configured
to your needs




Adjustment features



Backrest
adjustable upto 65°



Kneerest
adjustable upto 30°

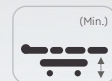

Lowest Height
position



Maximum
Height position



Trendelenburg



Reverse
trendelenburg

Technical Specifications Dimensions of all variants: Length: 224 cm +/- 0.05 cm; Width: 105 cm +/- 0.05 cm

	Acura Vital	Acura Standard	Acura Prime
Backrest	65° +/- 5°	65° +/- 5°	65° +/- 5°
Kneerest	30° +/- 5°	30° +/- 5°	30° +/- 5°
TR	NA	12° +/- 2°	13.5° +/- 1°
ATR	NA	8° +/- 2°	13.5° +/- 1°
Height Adjustment	Fixed Height: 56.1 cm	49 cm to 75 cm	44 cm to 77 cm
Safe working load	200 kg	200 kg	200 kg

Acura
Greater comfort,
greater efficiency.



Safety & comfort for patient.
High hygiene & ease of use for caregivers.
Premium aesthetics & a choice of options
for Hospital management.

Healthcare spaces today are expected to deliver at the highest standards of comfort, safety, efficiency and aesthetics. With minute attention to detail, we have designed Acura to meet the needs of various healthcare stakeholders. Confidently moving towards responsive healing spaces.


The **finer** details



125 mm Castors* with diagonal locking:
Maneuverability is easier with 125 mm castors, making it easy for caregivers to mobilise patient, quickly.



Bumpers with excellent shock absorbing property:
Keep the bed safe, provide protection against collisions during shifting of bed.

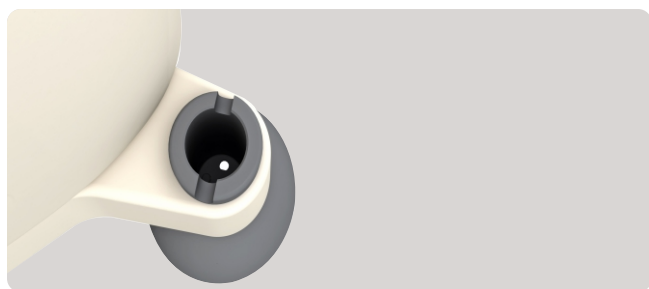


Provision for Lite IV Pole:
Provision for IV pole on all four corners of bed.



Urine Bag Holder:
MS hooks have been provided as urine bag holders on either side of bed, for ease of accessibility.




Emergency CPR Lever:
Helps achieve flat position of backrest in case of emergency, from either side of the bed.

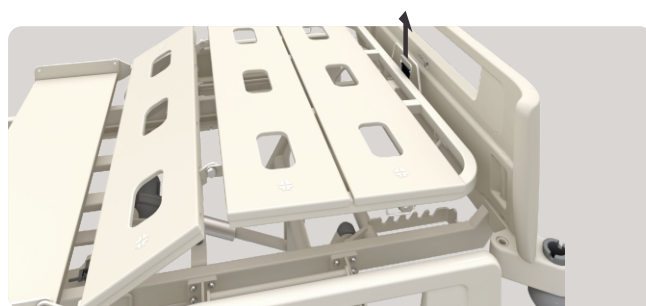

Leg Elevation:
Simple single hand operation to achieve leg elevation in case of DVT.

* Twin wheel & Single Wheel Castor options available.

Safety

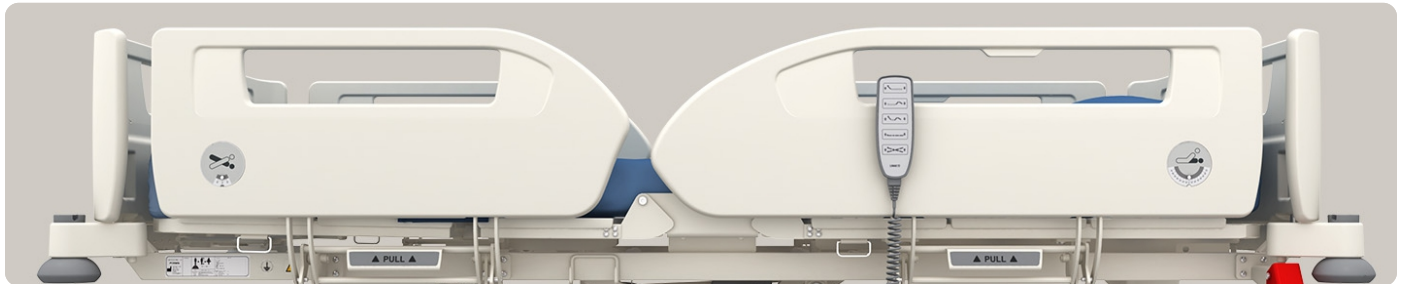

Full Coverage Safety Side Boards: Patient safety is at the core of our design. The full coverage sideboards provide utmost safety and make patient handling easier.



Zero Entrapment zones: IEC - 60601-2-52 Safety Standards compliant, ensures body parts of caregivers and patient are safe at all times.




Separate angle indicator for headboard and footboard: Caregiver has access to angle readings of bed at all times, helps in achieving safe position for patient.

Hygiene



Removable Plastic Headboard & Footboard: For easy cleaning or in case of emergencies.

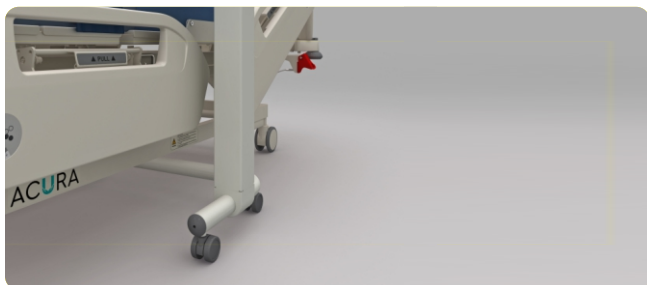

Adequate Under-bed clearance: The optimum Under-bed clearance (>15 cm) ensures unobstructed placement of medical equipment under the bed; ease of bed maintenance.




Removable Lying surfaces: The removable PP lying panels make cleaning and maintaining high hygiene standards easier.

Side Boards

Our wide range of side board and side rails options gives you the opportunity to create safe healthcare spaces with new age aesthetics. Choose the ones best suited to your hospital environment and patient needs.


Blowmould Sideboards

Provide full coverage with drop down mechanism. The sideboards, with superior surface texture and finish, are **rotatable** with back rest.

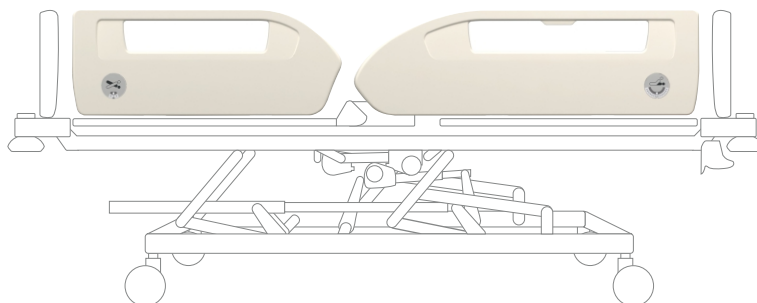


Aluminium Sideboards

Provide full coverage with drop down mechanism. The sideboards with classic Aluminum touch in matte finish give the bed a novel look.

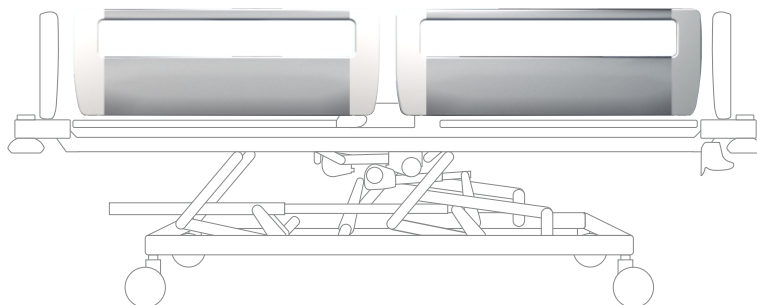


Aluminium Siderails

Provide 3/4th coverage with drop down mechanism. These siderails provide a distinct look to the bed, while meeting the standards of patient safety.

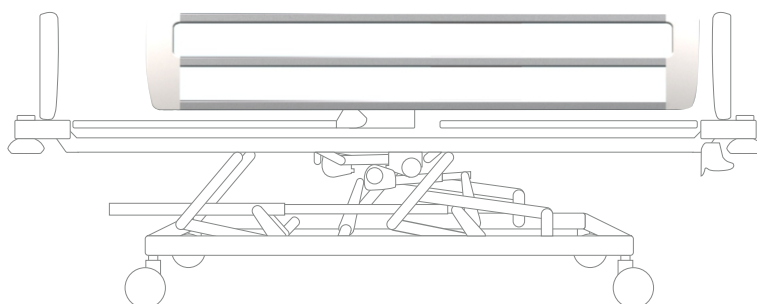

MS Siderails

The MS powder coated collapsible siderails provide optimum coverage with swing side mechanism. Lowering the siderails vertically is easy which makes it ideal for use in confined spaces.

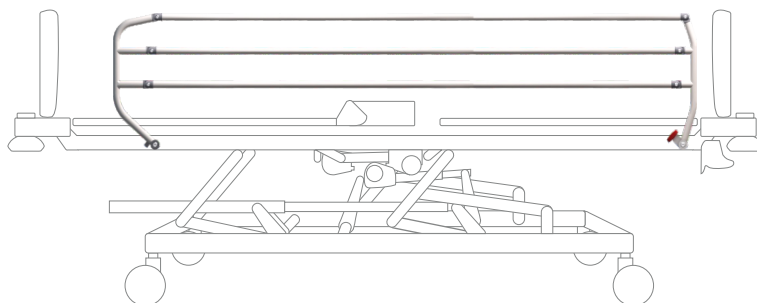

Accessories




Actuator

Attendant Remote

Control Box

Actuator Kit: The retrofit Actuator Kit is used to upgrade the bed features from manual to motorised.

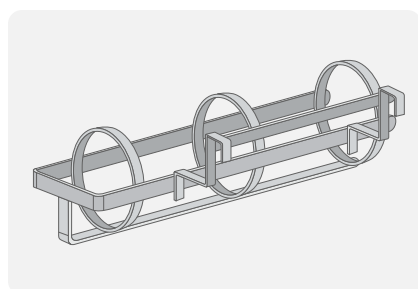


Oxygen cylinder holder

Supports mobile respirator with spontaneous breathing mode.




Lite IV Pole

Ergonomic SS IV pole, easy to use at any bed corner.

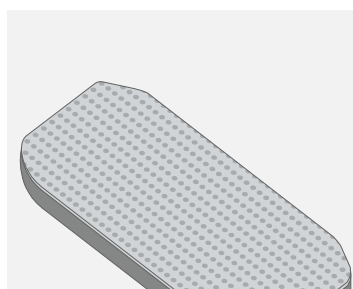

Mattress

Patient responsive mattress is the ideal match, facilitating comfort and quick recovery.

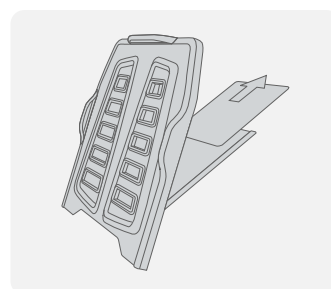

X-ray permeable back

Eliminates the need to shift patient for x-ray purposes, reduces stress on patient and caregiver.

World Class Standards & Certifications

Patent No. 202021004945

IEC 60601-1 Medical Electrical Equipment: General requirements for safety and essential performance.

IEC 60601-2-52 Medical Electrical Equipment: Particular requirements for safety and essential performance of medical beds.

IEC 60601-1-2 Electromagnetic compatibility. (Requirements & Tests)

IEC 60601-1-4 General Requirements for Safety: Programmable electrical medical systems.

ISO 14971 Medical Devices: Application of risk management to medical devices.

ISO 13485:2016 Medical Devices: Quality Management Systems - Requirements for regulatory purposes.



AVAILABLE IN OVER 600 LOCATIONS ACROSS INDIA

Put some play into your work by visiting
godrejinterio.com



An initiative by Godrej Interio, the first Indian furniture company with **GREENGUARD** certified products

MODULAR FURNITURE • DESKING • SEATING • STORAGE • INTERIORS • LAB • MARINE • HEALTHCARE • HOME

Local Address

National

North: Delhi, Chandigarh, NCR, Lucknow
South: Chennai, Bengaluru, Hyderabad, Kochi
West: Mumbai, Ahmedabad, Bhopal, Pune
East: Kolkata, Bhubaneswar, Guwahati, Patna

In keeping with the Godrej policy of continuous development and improvement, the design, dimension and specification may be updated without prior notice or obligations. Form, Colour and Finish of product in this document are representative of design intent and may vary from actual. Colour options depicted in this catalogue are as close to what modern printing processes allow. All dimensions are in 'cm' and are subject to commercial tolerance. Our products are meant for indoor use only.

Godrej & Boyce Mfg. Co. Ltd.

Godrej Interio, Pirojshanagar, Vikhroli (W), Mumbai – 400 079.
Toll Free Number: 1800-267-1122. Fax: +91-22-6796 1503
Email: interiohospital@godrej.com
Website: www.godrejinterio.com

Follow us on



/godrejinterio



/godrej-interio



/godrejinterio



/godrejinterio4u